Lecoxen®

SPRAY

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Lecoxen®

SPRAY

Isotonic sea water solution with trace elements for irrigation and cleaning of wounds

Lecoxen® Spray is a medical device intended for topic treatment of acute and chronic wounds in order to clean and irrigate them. Thanks to a jet of isotonic sea water solution rich in trace elements, it is effective in removing dirt, foreign bodies, necrotic residues and exudate from the wound and it facilitates the control of the wound microenvironment and wound healing. Ideal in first aid situations. The bottle can be used in any position. Disinfectant free.

Directions for use and dosage: to be used for wound cleansing. Before applying the dressing, adjust the power of the water jet putting near or removing the nozzle until you reach the most suitable distance for doing so. Do not position the nozzle in close contact with the wound. Remove possible excess solution with sterile gauze. The treatment with Lecoxen® spray is recommended before treating the wound with any clinical protocol.

Warnings: do not use on wounds which have breached the dermis and can only heal by secondary intent. For personal use. Wash hands thoroughly before use. It is recommended to wear disposable gloves if the product is used by third parties. Keep away from heat sources, hot surfaces, sparks, open flames or other sources of ignition. Do not smoke. Protect from sun rays. Do not expose to temperatures exceeding 50°C. Do not pierce or burn, even after use. Keep the product in a safe place. Keep out of reach of children. Do not use if the package is opened or damaged. Do not use after the expiry date shown on the bottom of the bottle.

***Lecoxen® spray** is an isotonic solution of sea water and demineralised water without preservatives and disinfectants ideal for wound cleaning. It is the primary and fundamental step for the management of wounds of different etiology. Indeed, it should be considered that the first phases of the treatment of wounds include cleaning and disinfection which are to be considered two different methods.*

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MEDWORLD
Innovative Healthcare Solutions

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